

ENG COMPRESSOR NEBULIZER LD. Instruction Manual.

POL INHALATOR KOMPRESOROWY LD. Instrukcja obsługi.

HUN KOMPRESSOROS INHALATOR LD.SOROZAT. Használati útmutató.

ROLI APARAT DE AEROSOLI CU COMPRESOR LD. Manual de instrucțiuni.

BGR КОМПРЕСОРЕН ИНХАЛАТОР LD. Ръководство за експлоатация.

1. FIGURES | ILLUSTRACE | RAZKON | FIGURI | ФІГУРИ

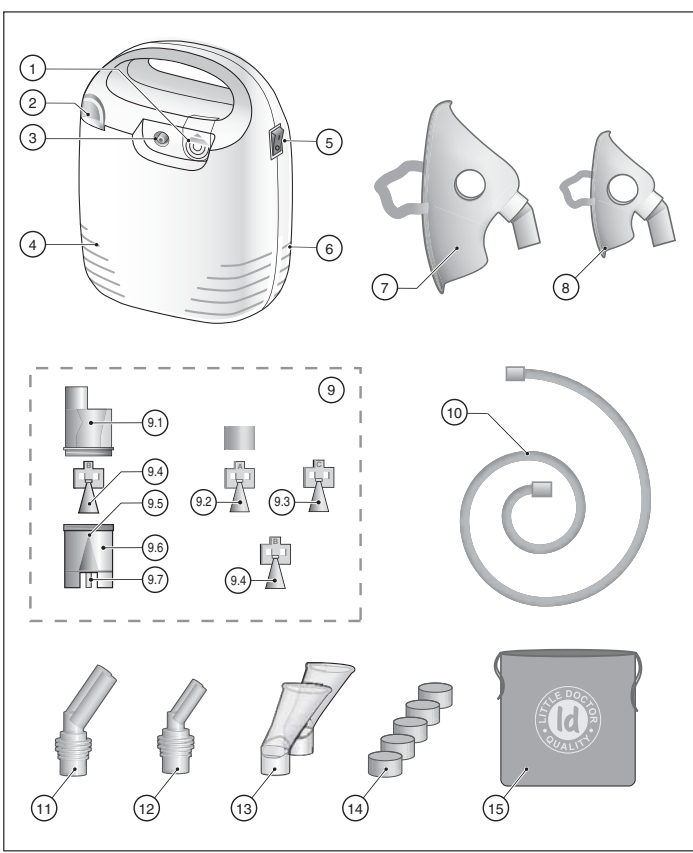


fig.1 | rys.1 | 1. ábra | fig.1 | рис.1

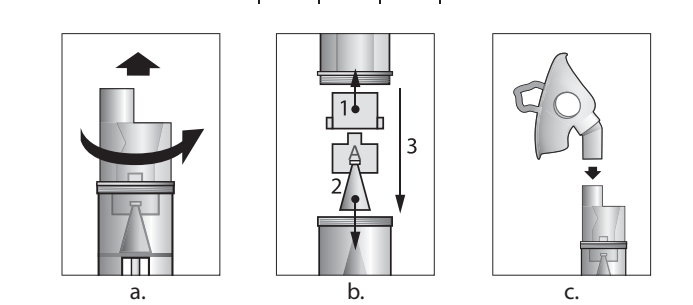


fig.2 | rys.2 | 2. ábra | fig.2 | рис.2

Baffle "C" (red) Rozryplacz «C» (czerwony) Adagoló "C" (piros) Disperser "C" (rosu) Пульверизатор "С" (червен)

mainly the upper respiratory tract nebulizacja górných dróg oddechowych főleg a felső légutakhoz preponderant tracheal respirator superior predimno za gorintje dihalatni pľtišha

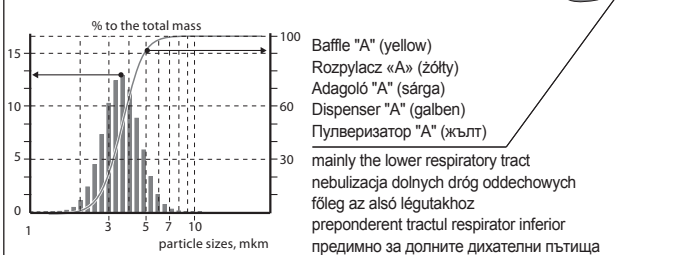


fig.3 | rys.3 | 3. ábra | fig.3 | рис.3

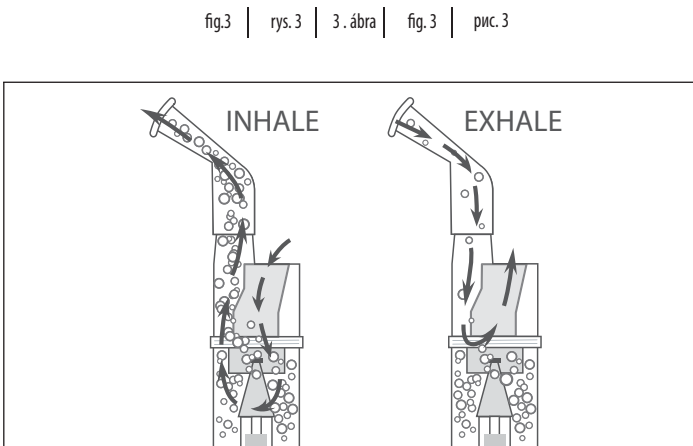


fig.4 | rys.4 | 4. ábra | fig.4 | рис.4

2. MAIN PARTS AND COMPONENTS (Fig.1)

№	NAME	DESCRIPTION
1	Socket for the Filter	Position the air filter for nebulizer.
2	Nebulizer holder	Holding for the nebulizer.
3	Connector of compressor	Holder for compressor for connecting an inhalation tube.
4	Compressor	Nebulizer compressor unit to create air pressure.
5	Power Switch	Turnbler I/O – switch on/switch off power.
6	Intake	Holes for air intake to cool the compressor.
7	Inhalation mask LD-N041	Adult inhalation mask. Expendable material.
8	Inhalation mask LD-N040	Child inhalation mask. Expendable material.
9	Nebulizer LD-N05	Chamber for inhalation of aerosol from a solution. Expendable material.
9.1	Upper part of the nebulizer	Part of the aerosol chamber. Place of connection to the sleeve.
9.2	Baffle "A" (Yellow) LD-N001	Inhalation baffle (mainly the lower respiratory tract). Consist of two parts: cone and cylinder. Expendable material.
9.3	Baffle "C" (Red) LD-N003	Inhalation baffle (mainly the upper respiratory tract). Expendable material.
9.4	Baffle "B" (Blue) LD-N002	Inhalation baffle (universal). Expendable material.
9.5	Nozzle	Cone-shaped nozzle to create a thin air jet.
9.6	Lower part of nebulizer	Part of the aerosol chamber. (for inhalation solution with a seat for the bumpy).
9.7	Air tube connector	Socket for connecting an inhalation tube.
9.8	Air tube LD-N051	Inhalation tube. Expendable material.
9.9	Adult Nasal Nozzle LD-N058	Adult nozzle for inhalation through the nose. Expendable material.
10	Child Nasal Nozzle LD-N059	Child nozzle for inhalation through the nose. Expendable material.
11	Mouthpiece LD-N022	Inhalation mouthpiece. Expendable material.
12	Air filter LD-N055	Filter inhalation. Expendable material.
15	Bag	Accessories storage bag

3. NEBULIZER THERAPY – WHAT IS IT?

Nebulizer is a device for formation and spraying of aerosol. The word "nebulizer" is derived from the Latin word "nebula" (fog, cloud) and was first used in 1874 for a device that turns a liquid substance into an aerosol for medical purposes. One of the first portable "aerosol apparatuses" was created by J. Sales-Girons in Paris in 1859. The first nebulizers were used as steam jet energy sources and were applied for inhalation of vapors of resins and antiseptics by tuberculous patients. Presently, the term "inhaler" is often used instead of "nebulizer". The purpose of the nebulizer therapy is to quickly deliver to the respiratory passages a therapeutic dose of a preparation in aerosol form. Continuous supply of aerosol allows, within several minutes, creating high concentration of a medicine in the upper and lower respiratory passages and lungs, with low probability of any by-effects. Respectively, effective bronchodilation (bronchi expansion) is reached, and the need for hospitalization is eliminated or the hospital stay is reduced. Little Doctor International (S) Pte. Ltd. offers you to use inhaler LD-215C, whose distinctive features are the possibility to use a wide range of medicines, low inhalation solution residual volume, and reliable and simple use. We thank you for your choice.

4. GENERAL INFORMATION

Compressor nebulizer LD-215C is designed for treating the diseases of respiratory passages and lungs by medicine solution aerosols in clinic. This Instruction Manual is designed to assist the use with safe and effective operation of the Compressor Nebulizer LD. Use this Device according to the rules described in this Manual. Operate the Device only as intended. Do not use the device for any other purposes. Read and understand the whole Instruction Manual. Functionally, the device consists of an air compressor and nebulizer (aerosol formation chamber). The air compressor, on/off power switch and air filter are united in one casing. From the air compressor, the compressed air is fed through a pipe to the nebulizer, where aerosol is formed. For cooling the compressor, air is force-feed into the device's casing.

5. INDICATIONS FOR USE

The LD compressor nebulizer is designed for the treatment of respiratory diseases such as rhinitis, pharyngitis, laryngotracheitis, acute and chronic bronchitis, bronchial asthma.

6. CONTRAINDICATIONS

The LD compressor nebulizer is contraindicated for use if there are malignant neoplasms, systemic blood diseases, severe general exhaustion, stage III hypertension, pronounced atherosclerosis of the cerebral vessels, diseases of the cardiovascular system in the stage of decompensation, nosebleeds or a predisposition to nosebleeds, hemoptysis, fever (body temperature above 38 °C), active pulmonary tuberculosis, acute pneumonia, hypertrophy of the mucous membranes of the respiratory tract, tonsillitis, pleurisy, epilepsy with frequent seizures, hysteria with severe convulsive seizures, psychosis with symptoms of psychomotor agitation, alcohol or drug intoxication, individual intolerance to procedures, general severe patient status. If the patient has recently undergone dental surgery or if the patient is undergoing treatment related to problems in the mouth or throat, it is necessary to consult a doctor before use of the device.

7. SIDE EFFECTS

The use of the LD compressor nebulizer has no side effects if used with the observance of precautions.

8. PRECAUTIONS

- Important information!** For correct work of the nebulizer it is recommended to use all types of standard inhalation solutions in liquid form for nebulizer therapy produced by pharmaceutical companies.

- Solutions for inhalation should be prepared under sterile conditions using 0.9% sodium chloride as a solvent.**

- Attention!** Do not use running/municipal water for inhalations. Use only saline solution. The dishes in which the solution is prepared should be pre-disinfected by boiling.

- The type of inhalation (by mouth, using a mouthpiece or through the nose, using a mask or a nozzle), the frequency, the inhalation solutions used and the dosage must be determined by YOUR DOCTOR.*
- Place the device on a flat table surface for use.*
- To avoid overheating of the device, do not block the air inlet.*
- Children must use the device under the supervision of an adult.*
- If you are not using the device, disconnect it from the mains by pulling out the plug from the socket.*
- Do not touch the power plug with wet hands.*
- Do not place the device in water, under a water drain or in a shower stall. Do not use when bathing.*
- Do not touch the device if it has fallen into water. Unplug it immediately.*
- Make sure the power cord is not damaged before use.*
- The power cord must not touch hot or heated surfaces.*
- If the device does not work, see section "15. TROUBLESHOOTING AND ELIMINATION".*
- Use specified accessories specified for this model and described in this manual.*
- This device cannot be used for inhalation anesthesia and artificial ventilation of the lungs.*
- Do not insert foreign objects into the openings of the device.*
- The device is not intended for outdoor use.*

9. USING THE DEVICE

Preparing for inhalation.

- Important!** Before using the appliance for the first time it is necessary to make a full cleaning, as described in first paragraph «10. CARE, STORAGE, REPAIR AND DISPOSAL».

- Place the nebulizer in front of you on the table. Make sure the device is turned off (power switch is in position «0»), and the power cord is not plugged into the mains.
- Remove the top of the nebulizer by turning it counterclockwise (Fig.2 a).
- Set the desired baffle inside the nebulizer as baffle «B» (Blue), which is effective to affect the entire respiratory tract. For a more effective impact medicines on the upper respiratory tract, set, instead of the blue baffle, red baffle «C». For a more effective impact on the lower respiratory tract – baffle «A» yellow color, which consists of two parts (Fig.2b, figures showing the order of assembly). Graphics of the differential particle size distribution by mass for different baffles are shown in Fig. 3 (measured by Little Doctor International (S) Pte. Ltd.).
- Fill the bottom of the nebulizer inhalation solution. The dosage should not exceed the recommended by your doctor. The length of one treatment session is determined by the scratches on the scale. The maximum recommended volume of 10 ml.
- Attach the nebulizer at the top, turning it clockwise until it stops.
- Depending on the type of inhalation, using either a mouthpiece or nozzle or mask. The mask, mouthpiece or nose nozzle is connected directly to the upper part of the nebulizer (Fig.2.c). Hold the nebulizer vertically.

- Important!** Each patient is encouraged to use personal mouthpiece, a mask and / or a nozzle for the nose.

- Plug the power cord to an electrical outlet.
- Connect one end of the inhalation tube to the fitting of compressor, and others - to the fitting of nebulizer.
- Turn on the nebulizer, switching the power switch in position «1». NEBULIZER IS READY FOR INHALATION.

Performing the inhalation.

The length of one treatment session should not exceed 20 minutes. Consult your attending physician about the length of the inhalation procedure. You should always be calm and relaxed during the inhalation. Breathing should be slow and deep, so that the preparation could fill the lungs well and reach the deep portions of bronchi. Briefly hold your breath, and then exhale slowly. Do not attempt to breathe too rapidly. Make pauses if you need it you need it.

Breath-actuator method (Fig.4).

The special design of the nebulizer in the form of chambers connected in a certain manner provides different ways of air streams during inhaling and exhaling. 1. After obtaining the air stream with greatest aerosol concentration when inhaling and reducing aerosol loss when exhaling. The effectiveness of inhalation using the breath-actuator method is increased significantly.

Completing the inhalation.

When the inhalation time is used up and the inhalation time recommended by the doctor has expired, turn the device off by putting the tumbler in «0» position and unplug it. After inhalation, breathe fresh air for some time for better treatment effect. After each application of the device, the residual preparation should be removed out of it. Clean and wash the device as described in first paragraph «10. CARE, STORAGE, REPAIR AND DISPOSAL».

10. CARE, STORAGE, REPAIR AND DISPOSAL

- Before the first use, as well as after each use of the device, it should be cleaned and disinfected.

- Attention!** When cleaning and disinfecting, do not allow solutions of detergents and disinfectants to get inside the compressor block housing.

At home: Cleaning of the nebulizer, baffles disconnected from the nebulizer, masks, nozzles for the nose, mouthpieces and the tube is carried out by rinsing with warm water (with a temperature not exceeding 45 °C) with the addition of a 0.5% detergent solution. It is recommended to use detergents for washing children's dishes or washing children's clothes (laundry). The compressor block housing is cleaned by wiping with a gauze cloth soaked in a 0.5% detergent solution.

Disinfection of the nebulizer, baffles disconnected from the nebulizer, masks, nozzles for the nose, mouthpieces and tube is made by full immersion (soaking) in 3% hydrogen peroxide solution or in 3-6% vinegar solution (1 part vinegar to 3 parts distilled water). The disinfection time is 30 minutes. The initial temperature of the solution is 50 °C. The shelf life of the working solution is 1 day. Make enough solution to dip all of the parts. Disinfecting solutions must not be reused! Mouthpieces and nozzles for the nose can be boiled in distilled (or in boiled water for 10 minutes). After handling, wipe all parts of the appliance dry with a soft cloth.

In medical institutions: Cleaning and disinfection of the nebulizer, baffles disconnected from the nebulizer, masks, nozzles for the nose, mouthpieces and tube is performed using specialized disinfectants. Nozzles for the nose LD-N058 and LD-N059, mouthpiece LD-N022 can be boiled for 10 minutes. If necessary, the nozzles for the nose LD-N058, LD-N059 and LD-N022 can be sterilized in an autoclave at a temperature not exceeding 150 °C for 10 minutes. After handling, wipe all parts of the appliance dry with a soft cloth.

- Regularly check if the filter is dry and, if necessary, replace the filter. To replace the filter, open the filter slot by prying the filter slot cover with a screwdriver; install a new filter; close the filter slot IT IS RECOMMENDED TO CHANGE THE FILTER AT LEAST ONETIME A YEAR.

- The device must be protected from kids and direct sunlight.

- Do not store or use the device in the immediate vicinity of heaters and open flames.
- Protect the device from dirt.
- Do not allow the device to come into contact with aggressive solutions.
- If necessary, carry out repairs only in specialized organizations. Contact information is provided in section «16. MANUFACTURER AND AUTHORIZED REPRESENTATIVES».
- The service life of the compressor is 5 years from the date of manufacture. The service life of consumables is 1 year from the date of handover of the device to the consumer. After the expiration of the established service life, you should periodically contact the specialists (contact information is indicated in the section «16. MANUFACTURER AND AUTHORIZED REPRESENTATIVES») to check the technical condition of the device and, if necessary, to dispose of it in accordance with the current disposal rules in your region. The manufacturer has not established special conditions for disposal.
- The manufacturer does not establish specific standards for the frequency of preventive inspection and maintenance.

11. WARRANTY

The purchased device is warranted for 3 years from the date of sale. Faults detected during the warranty period will be repaired free of charge by a specialized service center within 21 business days from the date of device delivery to the service center. The cost for the defective device transportation are borne by Little Doctor Europe S.p. z o. o. (door-to-door service), except for faults that are not covered by the warranty. For the addresses of the warranty service providers, see MANUFACTURER AND AUTHORIZED REPRESENTATIVES or go to www.littledoctor.sg.

The party submitting the claim must properly prepare the device for shipment, protecting it from damage during transportation and enclosing a completed warranty card (with the seller's stamp, date of sale and seller's signature) and proof of purchase (invoice or receipt). The warranty does not cover damages resulting from inadequate protection of the device during transportation. The warranty period is extended for the time the device is under repair. Post-warranty repair of the device is performed at the user's expense. The warranty does not cover the following: consumables and accessories for individual use (buses, nebulizers, masks, mouthpieces, etc.); products with chemical, thermal or chemical damage, or damage caused by violations of the rules specified in the operating instructions; products with signs of modification, opening and/or repair by an unauthorized service center (by an individual); products that have been connected to unrecommended power supplies or used with unrecommended consumables; products that have been operated with non-repaired defects; products that are defective through no fault of the manufacturer, such as: actions of third parties, natural phenomena and natural disasters, animals, insects, foreign objects or liquids entering the device, etc.; internal and external contamination, scratches, cracks, scrapes and other mechanical damage to the equipment occurring during its operation; products with illegible or altered serial numbers or a damaged warranty seal; batteries, glass parts, light bulbs, as well as operations such as adjusting, cleaning and other maintenance of the product. The buyer has the right to replace the device with a serviceable one if an authorized service center finds a manufacturing defect that has been eliminated or after 3 successful repairs of the same malfunction.

12. COMPLETENESS

№	NAME	MODEL	QUANTITY, pc.
1	Compressor	—	1
2	Nebulizer (with inhalation baffle LD-N002)	LD-N015	1
3	Adult inhalation mask	LD-N041	1
4	Child inhalation mask	LD-N040	1
5	Adult nasal nozzle	LD-N058	1
6	Child nasal nozzle	LD-N059	1
7	Inhalation mouthpiece	LD-N022	2
8	Air tube	LD-N051	1
9	Inhalation baffle "A"	LD-N001	1
10	Inhalation baffle "B"	LD-N002	1
11	Inhalation baffle "C"	LD-N003	1
12	Air filter	LD-N055	5
13	Instruction Manual	—	1
14	Warranty Card	—	1
15	Packaging	—	1
16	Bag	—	1

13. TECHNICAL SPECIFICATIONS

Model	LD-215C
Type	Compressor Nebulizer

Power Consumption 190 VA

Nebulization rate, approximately Inhalation baffle «A» LD-N001 0.3 ml/min.* Inhalation baffle «B» LD-N002 0.4 ml/min.* Inhalation baffle «C» LD-N003 0.5 ml/min.*

Particle size (MMAD) Inhalation baffle «A» LD-N001 3.5 μm* Inhalation baffle «B» LD-N002 4.0 μm* Inhalation baffle «C» LD-N003 5.0 μm* Operation mode 20 minutes Cooling time 40 minutes Max capacity of nebulizer kit 10 ml Residual volume, not more 0.5 ml Noise level, approx. 51 dB*

Rating: AC230V, 50Hz type BF

Protection against electric shock Ambient temperature 10 °C to 35 °C Humidity 15 - 80% Rh Atmospheric pressure 86 to 106 kPa

Storage and transportation conditions for the device: Ambient temperature -20 °C to +40 °C Humidity 15 - 95% Rh Atmospheric pressure 50 to 106 kPa

Characteristics of the compressor unit: No-load performance, not less than Performance with connected air tubing and nebulizer, approx. 4 l/min Maximum pressure 210-400 kPa Operating pressure 60-180 kPa

Size 140x190x90 mm Net Weight, not more 1350 g

Year and month of production Indicated on the body of the device in the serial number as «AYMMXXXXXX», where YY is the year, and MM is the month of manufacture.

SYMBOL EXPLANATION:
C CARE Compliance with the Directive MDR (EU) 2017/745
I Important: Read the instruction
A Authorized Representative in the EU
M Manufacturer
P Keep from moisture
K Protection class II
BF type product
S Storage/operation temperature, humidity and atmospheric pressure limitation
SN Serial number
MD Medical Device
UDI Unique Device Identification
I Importer

This device manufacturing is certified according to international standard ISO 13485. Device comply with the requirements of European Directive MDR (EU) 2017/745, international standards EN 1041, EN 10601-1, EN 10601-2, ISO 14971.

Revision date of the present Manual is indicated on the last page as XX-XXXX-YYMM-NN, где YY is the year, MM is the year and NN is the month of revision. Technical characteristics may be changed without preliminary notification to improve the operation and quality of the product.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

14. TROUBLESHOOTING TIPS
The device is not turning on due to the following reasons.
- Lack of power supply in the network.
- Mismatch of the required power voltage.
- Poor contact between the plug and the power outlet.
- Blown Fuse. The fuse is installed inside the case. We recommend replacement of the fuse in a specialized repair organization.

The lack of aerosol formation can occur due to the following reasons.
- The baffle is not installed in the nebulizer.
- Lack of inhalation solution in the nebulizer. Add the required amount of solution to the nebulizer.
- Clogging of the nebulizer nozzle with sediments of the inhalation solution. Clean the nebulizer nozzle. During cleaning do not use metal objects that can distort the geometry of the nozzle.

- The inhalation tube is twisted. Straighten the inhalation tube so that nothing is blocking the flow of air into the nebulizer.

- Important!** Each patient is encouraged to use personal mouthpiece, a mask and / or a nozzle for the nose.

- Plug the power cord to an electrical outlet.
- Connect one end of the inhalation tube to the fitting of compressor, and others - to the fitting of nebulizer.
- Turn on the nebulizer, switching the power switch in position «1». NEBULIZER IS READY FOR INHALATION.

- Performing the inhalation.** The length of one treatment session should not exceed 20 minutes. Consult your attending physician about the length of the inhalation procedure. You should always be calm and relaxed during the inhalation. Breathing should be slow and deep, so that the preparation could fill the lungs well and reach the deep portions of bronchi. Briefly hold your breath, and then exhale slowly. Do not attempt to breathe too rapidly. Make pauses if you need it you need it.

Breath-actuator method (Fig.4). The special design of the nebulizer in the form of chambers connected in a certain manner provides different ways of air streams during inhaling and exhaling. 1. After obtaining the air stream with greatest aerosol concentration when inhaling and reducing aerosol loss when exhaling. The effectiveness of inhalation using the breath-actuator method is increased significantly.

Completing the inhalation. When the inhalation time is used up and the inhalation time recommended by the doctor has expired, turn the device off by putting the tumbler in «0» position and unplug it. After inhalation, breathe fresh air for some time for better treatment effect. After each application of the device, the residual preparation should be removed out of it. Clean and wash the device as described in first paragraph «10. CARE, STORAGE, REPAIR AND DISPOSAL».

10. CARE, STORAGE, REPAIR AND DISPOSAL
1. Before the first use, as well as after each use of the device, it should be cleaned and disinfected.

- Attention!** When cleaning and disinfecting, do not allow solutions of detergents and disinfectants to get inside the compressor block housing.

At home: Cleaning of the nebulizer, baffles disconnected from the nebulizer, masks, nozzles for the nose, mouthpieces and the tube is carried out by rinsing with warm water (with a temperature not exceeding 45 °C) with the addition of a 0.5% detergent solution. It is recommended to use detergents for washing children's dishes or washing children's clothes (laundry). The compressor block housing is cleaned by wiping with a gauze cloth soaked in a 0.5% detergent solution.

Disinfection of the nebulizer, baffles disconnected from the nebulizer, masks, nozzles for the nose, mouthpieces and tube is made by full immersion (soaking) in 3% hydrogen peroxide solution or in 3-6% vinegar solution (1 part vinegar to 3 parts distilled water). The disinfection time is 30 minutes. The initial temperature of the solution is 50 °C. The shelf life of the working solution is 1 day. Make enough solution to dip all of the parts. Disinfecting solutions must not be reused! Mouthpieces and nozzles for the nose can be boiled in distilled (or in boiled water for 10 minutes). After handling, wipe all parts of the appliance dry with a soft cloth.

In medical institutions: Cleaning and disinfection of the nebulizer, baffles disconnected from the nebulizer, masks, nozzles for the nose, mouthpieces and tube is performed using specialized disinfectants. Nozzles for the nose LD-N058 and LD-N059, mouthpiece LD-N022 can be boiled for 10 minutes. If necessary, the nozzles for the nose LD-N058, LD-N059 and LD-N022 can be sterilized in an autoclave at a temperature not exceeding 150 °C for 10 minutes. After handling, wipe all parts of the appliance dry with a soft cloth.

- Regularly check if the filter is dry and, if necessary, replace the filter. To replace the filter, open the filter slot by prying the filter slot cover with a screwdriver; install a new filter; close the filter slot IT IS RECOMMENDED TO CHANGE THE FILTER AT LEAST ONETIME A YEAR.

- The device must be protected from kids and direct sunlight.

* - Measured by Little Doctor International (S) Pte. Ltd.

POL

2. ELEMENTY GŁÓWNE URZĄDZENIA (rys. 1)

№	NAZWA	OPIS/PRZYZNACZENIE
1	Gniazdo filtra	Miejsce podłączenia filtra powietrza. Informacje dotyczące użytkowania i naprawy «rozdział <10. KONSERWACJA, PRZECYHOBYWANIE, NAPRAWA I UTYLIZACJA».
2	Uchwyt	Uchwyt na nebulizator.
3	Złącze kompresora	Złącze kompresora do podłączenia rurki inhalacyjnej.
4	Kompresor	Blok kompresorowy do wytwarzania ciśnienia.
5	Wtycznik	Wtycznik I/O – włączenie/wyłączenie zasilania.
6	Wloty powietrza	Otwory do ssania powietrza kompresora.
7	Maska inhalacyjna dla dorosłych LD-N041	Maska inhalacyjna dla dorosłych. Element ulegający zużyciu.
8	Maska inhalacyjna dla dzieci LD-N040	Maska inhalacyjna dla dzieci. Element ulegający zużyciu.
9	Nebulizator LD-N05 (z rozłącznikiem LD-N002)	Komora do wytwarzania aerozolu z roztworu inhalacyjnego. Element ulegający zużyciu.
9.1	Główna część nebulizatora	Część nebulizatora.
9.2	Rozryplacz inhalacyjny LD-N001	Rozryplacz «A» (żółty) – przeważnie do dolnych dróg oddechowych. Składa się z dwóch części: stożka i węża. Element ulegający zużyciu.
9.3	Rozryplacz inhalacyjny LD-N003	Rozryplacz «C» (czerwony) – przeważnie do górnych dróg oddechowych. Element ulegający zużyciu.
9.4	Rozryplacz inhalacyjny LD-N002	Rozryplacz «B» (niebieski) – uniwersalny. Element ulegający zużyciu.
9.5	Dysza	Otwór na kształt stożka do wytwarzania cienkiego strumienia powietrza.
9.6	Dolna część nebulizatora	Część nebulizatora (do roztworu inhalacyjnego z miejscem na rozryplacz).
9.7	Złącze nebulizatora	Złącze do podłączania rurki inhalacyjnej.
9.8	Rurka inhalacyjna LD-N051	Rurka inhalacyjna. Element ulegający zużyciu.
9.9	Nasadka do inhalacji przez nos dla dorosłych LD-N058	Nasadka do inhalacji przez nos dla dorosłych. Element ulegający zużyciu.
9.12	Nasadka do nosa dla dzieci LD-N059	Nasadka do inhalacji przez nos dla dzieci. Element ulegający zużyciu.
9.13	Ustnik do inhalacji LD-N022	Ustnik do inhalacji. Element ulegający zużyciu.
9.14	Filtr inhalacyjny LD-N055	Filtr powietrza do inhalatora. Element ulegający zużyciu.
9.15	Etui na akcesoria	Etui do przechowywania akcesoriów.

3. CZYM JEST TERAPIA AEROLIZOWANA?

